

WHATS IN THE BOX

The MonBaby Retail Bundle includes

One (1) MonBaby Smart Button (battery included)

One (1) Button Enclosure

User Manual

FASTENING THE SMART BUTTON

1. Place the Button Enclosure on the inside of your baby's pajama or onesie, with the convex side facing your baby.
2. Align the Smart Button on the outside of your baby's pajama with the Button Enclosure, and press it into the enclosure so that it is fastened to the clothing.

IMPORTANT: For best performance, fasten MonBaby in a location nearest to the center of your baby's chest.

DOWNLOADING THE APP

The MonBaby app can be easily downloaded by going to MonBabyApp.com on your mobile phone browser, where you will be directed to the iTunes or Google Play app listing.


You can also scan the QR Code below:



CONNECTING TO THE APP

Before opening the MonBaby app, it is best to have Bluetooth already enabled on your device, to make the setup process as quick as possible. You can do this by going to your phone's 'Settings' app and finding the Bluetooth option.

When you first open the MonBaby app, you are taken to the Connection Setup screen, which lets you connect the Smart Button to the App.

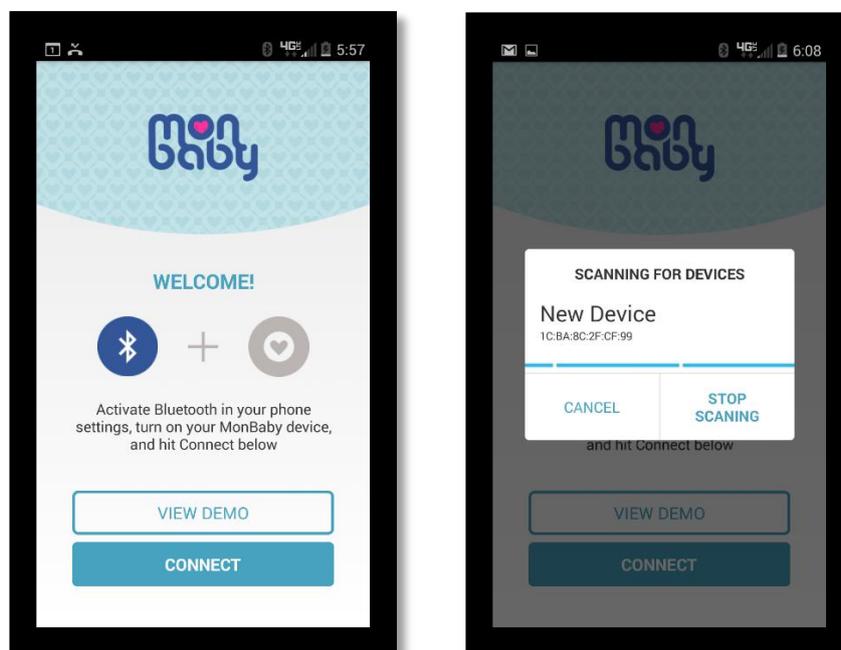

If Bluetooth is already enabled on your device, the Bluetooth icon on the left should appear blue and you can proceed to the next step. If the Bluetooth icon is grey, you can tap it to enable Bluetooth on your phone.

Once the Bluetooth logo is blue, you are ready to connect. Tap the **Connect** button at the bottom of the screen, and you should see a pop-up with the detected MonBaby devices listed. A new MonBaby will be listed as *New Device*. Tap on the device name to complete the pairing. If it is your first time pairing the device, you will be prompted to enter your baby's name for a personalized experience.

Once pairing is complete, you will be taken to the Dashboard. If you are unable to connect the Smart Button to the App, please see our Troubleshooting section or contact customer support.

REMOVING THE SMART BUTTON

To remove the MonBaby Smart Button, place your hand under your baby's pajama and, using your index finger, push the smart button out through the hole in the center of the Enclosure. Be sure to have your other hand on the Smart Button to collect it after you push it out, and remove the enclosure as well. The Smart Button can be stored inside of the Enclosure, and is put into standby mode automatically when not in use.

REPLACING THE BATTERY




Occasionally, you will need to replace the MonBaby Smart Button battery. You can always check the battery level by opening the MonBaby App and checking the top center of the Dashboard screen. To replace the **CR2032 COIN CELL BATTERY** in the Smart Button, follow the steps below:

1. Remove the Smart Button from the Enclosure
2. Turn the Smart Button so that the bottom (the side without the LED light) is facing up

3. Place your thumb on the heart engraving, press down, and slide your thumb down to remove the battery door
4. Use your finger to slide out the existing battery
5. Insert the new battery with the positive (+) side facing up and the negative (-) side touching the green circuit board
6. Slide the battery door back on to the Smart Button until it clicks into place

USING THE APP

Once you have your MonBaby Smart Button fastened and connected to the app, you can use the app to monitor your baby's sleeping habits. You can see the details of the app features below.

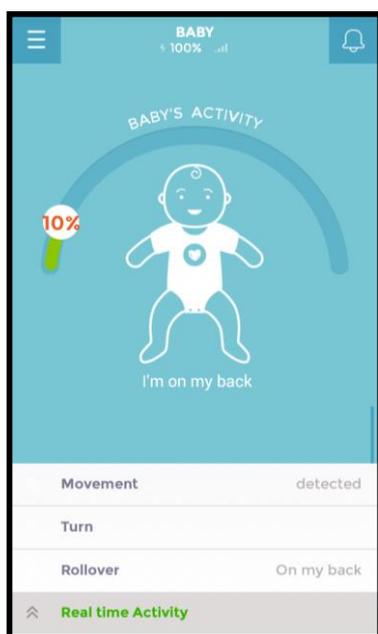

DASHBOARD

ACTIVITY: This bar shows you the movement level of your baby


STATUS VISUAL: This image represents whether your baby is sleeping on its stomach or back, and status indicates the baby's current status

MOVEMENT: 'Detected' or 'Idle'

TURN: The angle of the last detected turn

POSITION: This tells you if your baby is on its back or on its stomach

REAL-TIME ACTIVITY: The Real-Time Activity chart shows you your baby's movement activity for the past minute

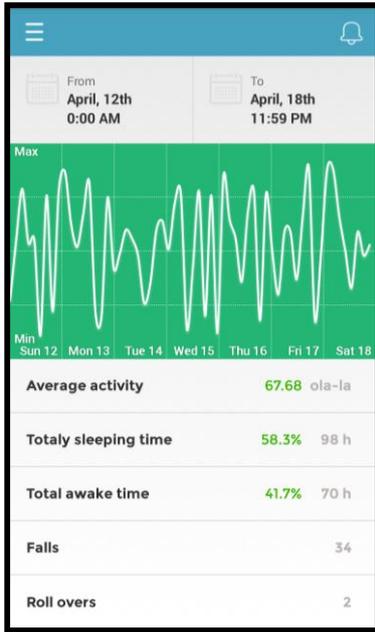


GRAPHS

DATE SELECTION: Select the beginning and end date and time to be represented in the *Activity Chart* and *Stats* below

ACTIVITY CHART: This bar shows you the movement level of your baby

STATS: You can see *Average Activity*, *Total Sleeping Time*, *Total Awake Time*, and *Roll Overs* for the date/time period you selected.

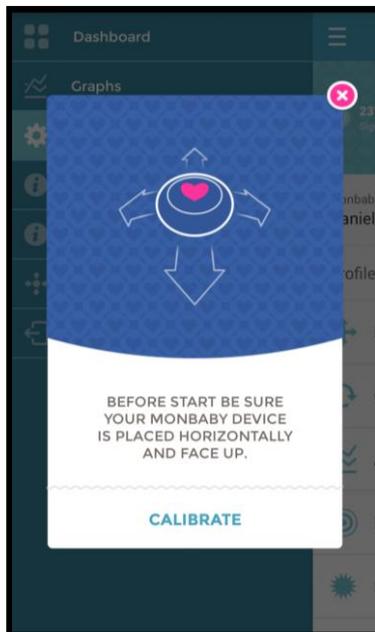


CALIBRATE

The calibrate screen should pop up after you fasten the Smart Button to your child's pajama and connect the Smart Button to the app.

You can always re-calibrate by selecting *Calibrate* from the slide-out menu.

VERY IMPORTANT: For best results, place the Smart Button near the center of your child between its chest and abdomen, with the baby lying on its back in his or her bed.

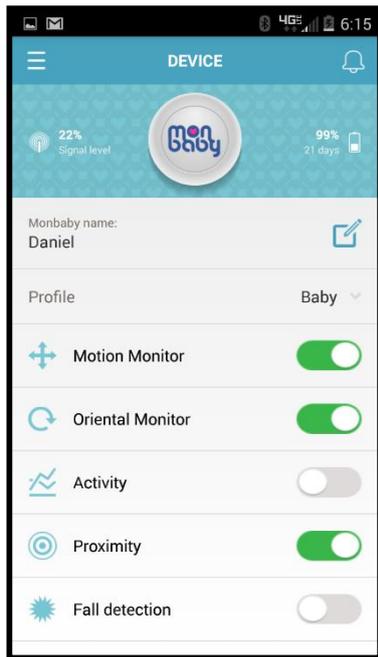

SETTINGS

SIGNAL LEVEL INDICATOR: The percentage of signal you have between your smart phone and the Smart Button

BATTERY LIFE INDICATOR: The percentage of battery remaining in the Smart Button and estimated remaining usage time

MONBABY NAME: View and edit the name associated with the Smart Button device

PROFILE: Select between 'Baby' (single) or 'Twins' (double)

MOTION ALERT: Alerts you if there is a suspected stoppage in breathing (absolutely no motion)

ORIENTATION ALERT: Alerts you if the baby rolls over on to its stomach

ACTIVITY ALERT: Alerts you if the baby wakes up out of its sleep

PROXIMITY ALERT: Alerts you if the Smart Button (and your child) is out of range of your smart phone device

FALL DETECTION: Alerts you if the baby has fallen

SHARE STATS: Share stats with your physician or specialist

TROUBLESHOOTING & SUPPORT

If you are having any issues, please check out the support page or our FAQ on www.monbaby.com, or e-mail our support team at support@mondevices.com.